VULPES GOODS

BABY CARE

MILKCOLLECTOR

MANUAL

ENGLISH

3

User Guide

4

Characteristics and Application

8

NEDERLANDS

11

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12

Kenmerken en Toepassingen

16

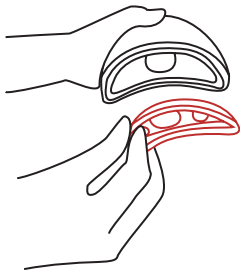
VULPES GOODS®

BABYCARE


ENGLISH

USER GUIDE

When receiving the collector, make sure you install the parts correctly to prevent the product from leaking and working less efficient.

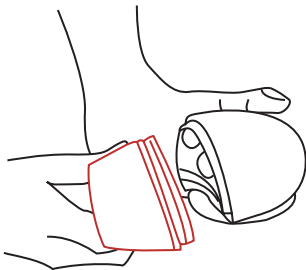
1. Install the plastic fixing bracket under the silicone upper part.

2. Install the upper set on the milk storage chamber.


After installing, make sure you keep the breast shield of the collector close to the breast to make it fit the breast closely.

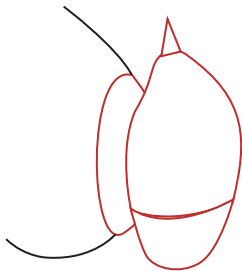Note: make sure the duckbill valve is upward.


Start sucking milk slowly by pressing the middle of the collector.

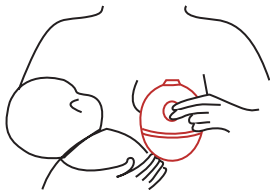
Note: Press at the “pump” part for the best suction.

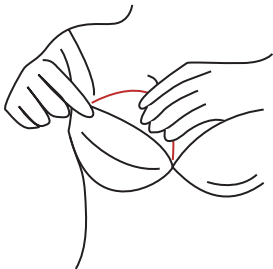Put on the bra and fix the collector in position.


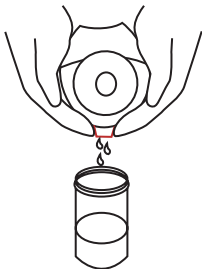
When done, take out the milk carefully by pressing the duckbill valve and pouring into a cup or storage bag

When done, make sure you clean the collector before storing it. Separate all the parts and clean them under rinsing water, remove all the residue.

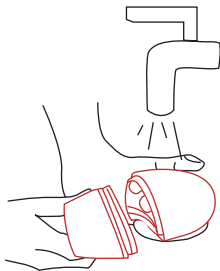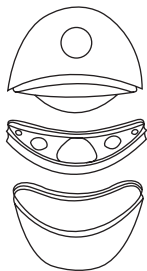Make sure you check out the Characteristics and application for the right way of cleaning.

CHARACTERISTICS AND APPLICATION

- After each usage, make sure you clean the collector before storing it.
- The collector needs to be pressed manually, it will not work automatically.
- Keep the collector away from open fire.
- Do not use chemicals, soaps or other agents that may contain potentially harmful substances. These may pose a hazard to the product and user.
- Do not put the parts in the dishwasher. This may damage the flasks and is not covered by the warranty.
- Read the manual carefully. This will ensure that the pump is properly installed.

- Store the collector in a suitable place. This means that the product is not in contact (while storing) with moisture, water or dust and is out of reach of children. We recommend storing the flask in the product box.
- Use the collector in a comfortable place. A comfortable situation facilitates pumping and can increase yields.
- Keep the pump clean and free of milk residue. This promotes longevity and quality of pumping.
- Keep all extra pouches and containers out of the packaging. You can always store the parts in here if you do not use the breast pump for a longer period of time.
- Massage your breasts briefly before pumping, this can increase the yield and make you feel more

comfortable while pumping.

- Boil the collector in water for three to five minutes. Make sure the water is deep enough to completely cover the parts. Watch the stirring accessories during cooking to avoid deformation due to uneven heat.
- We do not recommend microwave or steam sterilisation. Keep in mind that some steam and microwave sterilisers reach high temperatures, which can melt plastic parts.
- We do not recommend ultraviolet sterilisation.
- Let the parts air dry or pat dry with kitchen paper. After all, this does not give off dust particles that are

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