VULPES GOODS
BABYCARE

ELECTRIC BREAST PUMP

PRO V2

**MANUAL • HANDLEIDING
ANLEITUNG • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH

3

Product Information	4
Warnings	18
Cleaning & maintenance	21
FAQ	24
Product parameters	25

DEUTSCH

26

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
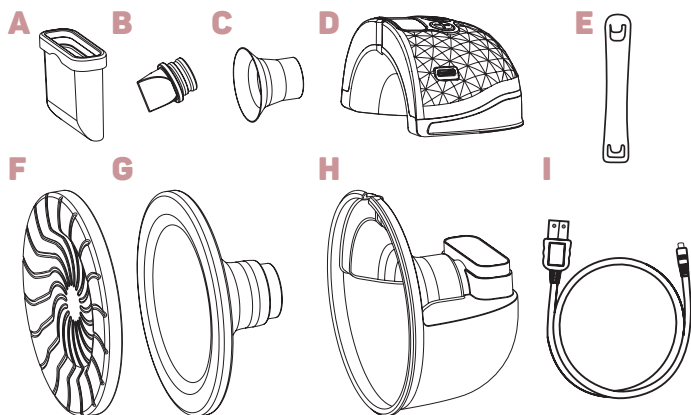
Assembly of the breast pump

When you receive the pump, the 24mm breast shield reducer is already inserted in the 28mm breast shield. Should you wish to clean the pump extra before use, it is useful to know how to assemble it correctly next. Assembling the breast pump and all its parts correctly is crucial for optimal use. Follow the steps carefully and make sure each part is correctly positioned.

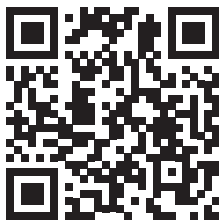When you scan the QR code, you will be directed to the instructional video. This will show you the best way to assemble, pour, and clean the pump. On the next page, you will find step-by-step instructions on how to assemble the parts and what to pay attention to.


Note: Before using the breast pump, watch the video for proper installation, cleaning, and maintenance. Always carefully watch the video and read the manual. This will help you use the breast pump effectively.

Step 1: Installation of gas storage


The gas storage is placed at the back of the collection tank.

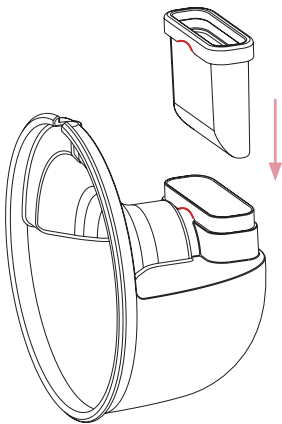Make sure that the bottom side is correctly mounted with the **curve** towards the pump.


Step 2: Placement of the valve

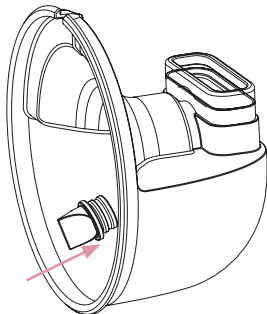The valve is placed in the receptacle of the flask. Make sure the valve is fully pressed on. This prevents the milk from flowing back.


Step 3: Placement of the breast shield

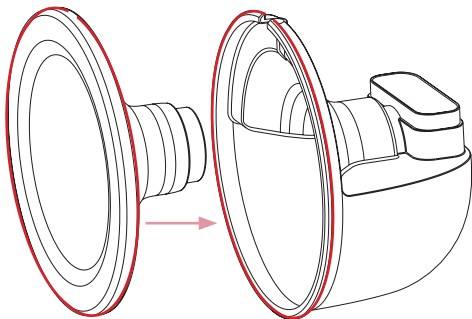The breast shield is placed on the collection tray. It is crucial that the breast shield is fully compressed and fits over the **edge** of the collection container.


Step 4: Placing the engine

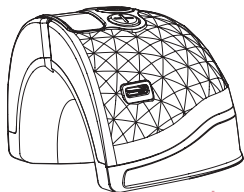
The engine attaches to the back of the container. Press the engine firmly onto the gas storage.

Step 5: Inserting the Breast Shield Funnel

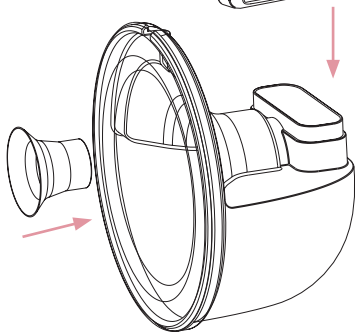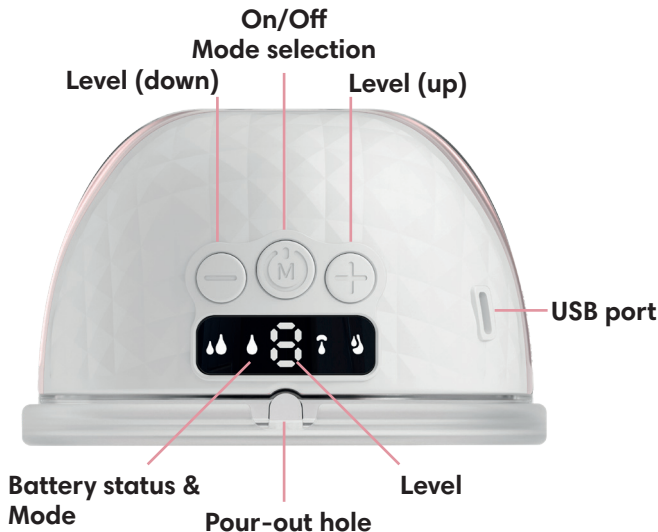Choose the right size breast shield reducer and insert it into the breast shield. Press the breast shield reducer firmly to prevent leakage.

On/off & mode selection

Press and hold the button for 2 seconds to turn the engine on. Press again to select the mode. Press and hold the button for 2 seconds to turn the engine off.

Level + or -:

Use this to select the level with which to pump. Depending on the selected mode, 1 - 9 levels are available. Each level up increases the suction power. (Always start at level 1 and build up to the desired level.)

USB Port

Use the charger to charge the engine. Insert and remove the charger carefully. Do not use excessive force, this may cause damage.

Level

The display shows the current level; each level increases the suction power. Using the + and - buttons, the level can be changed

Battery status:

If the light of the selected function flashes, it means that the battery is low and needs to be charged. During charging, the display lights flash. If the display lights are constantly on, it means that the battery is fully charged and the pump can be used again.

(Battery life depends on modes and levels used)

Mode

The display shows the current mode, each mode has a different function. Using the M button, you can change the mode. The engine has a built-in timer. After 30 minutes of pumping, the pump automatically goes into "sleep mode". You can switch the breast pump on again after this.

Memory mode

The breast pump remembers the last selected level in the mode. If you switch between modes, the pump will continue at the last stopped level.

Massage/stimulation mode (9 levels)

It is advised to massage the breasts for 2 minutes before pumping. This promotes yield and ease during pumping.

Expression/pump mode (9 levels)

Use expression/pump mode to pump fully. Each level offers different strengths while pumping.

2-in-1 mode (9 levels)

Can be very helpful for sensitive breasts and supports milk yield in short time.

Dual frequency mode (9 levels)

Can be very helpful for shortened or retracted nipples.

Note: Build up the levels slowly in each mode. Start at level 1 and build up to the desired level.

Breast Shield Reducers:

Choosing the correct breast shield reducers is important for optimal results. It ensures a better fit and prevents leaks. The table below will help you make the right choice.

Nipple diameter for pumping (mm)	Nipple diameter after pumping (mm)	Suggested size (mm)
14-15-16	16-17-18	19mm
17-18-19	18-19-20	21mm
20-21-22	21-22-23	24mm
24-25-26	25-26-27	28mm

Placement of the breastpump

Lean slightly forward and center the nipple in the breast pump. Ensure that the pump fits snugly against the breast.

Once the pump is in position, remain upright with your upper body. Ensure that the pump is positioned higher, and then begin expressing milk.

Once you're done expressing milk, lean forward to detach the breast pump. Lean slightly forward to avoid spilling any milk.

Bra - strap

Adjust the length of the bra with the straps to customize the fit. This can make pumping more comfortable.

The end of the bra strap can be attached to the nursing bra.

The length of the bra strap can be adjusted by pulling the bra strap tighter.


Pouring the milk

Pouring out the milk is easy via the pouring hole at the top of the collection tray. Always pour carefully without the engine sitting on the collection tray. This reduces the risk of moisture and milk damage.

The illustrations below show step-by-step how to prepare the pump for optimal pouring.


Step 1: Remove the motor

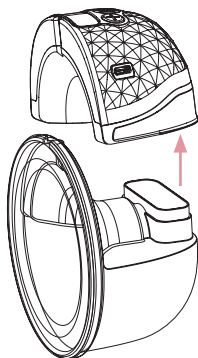Remove the engine first. Do this carefully to avoid leaks. Leave the gas storage in place. Otherwise there is a risk of a leakage.

Step 2: Tilting and pour

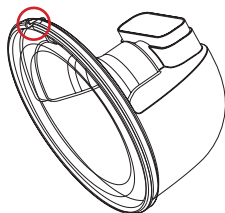Hold the breast pump by the breast shield, this is the easiest way to pour and gives control. As pouring improves, turn the flask further and further until the receptacle is empty.

Connecting to the app

To make pumping even more enjoyable, you can pair the PRO V2 with the EasyPump app via bluetooth. From the app, you can change the mode but it is also possible to keep track of the yield. Follow the steps below to use the app correctly.

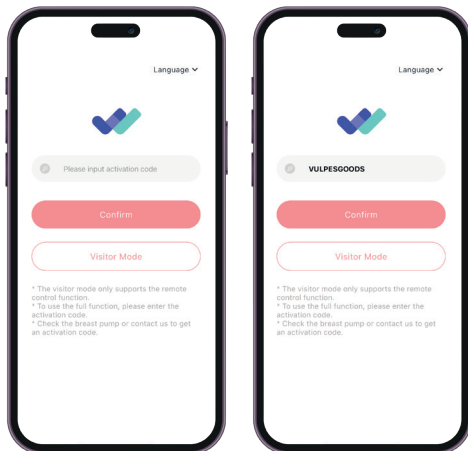
Using the app

- The app can only be used if the breast pump is on and connected.
- Turning the breast pump on from the app is not possible, switching it off is. Controlling the breast pump is possible via the app after switching it on.
- A minimum of 5 seconds must have been pumped, otherwise the session cannot be closed.
- Always select the right pump to operate. We recommend you rename the breast pumps.
- When pumping with two pumps and off is pressed. Will stop only the selected breast pump.
- If pumping is resumed from pause mode, there may be a second delay after pressing play.

Step 1: Open the app and connect

Open the app and enter VULPESGOODS as the activation code to log in.

Note: The code is case-sensitive.


Step 2: Log in or create an account

After entering the activation code, you can create an account:

Enter your email address and click Send to receive a verification code. Check your email and enter the received code in the Code field.

Set a password to complete your account setup. You can now log in using this email address and password.

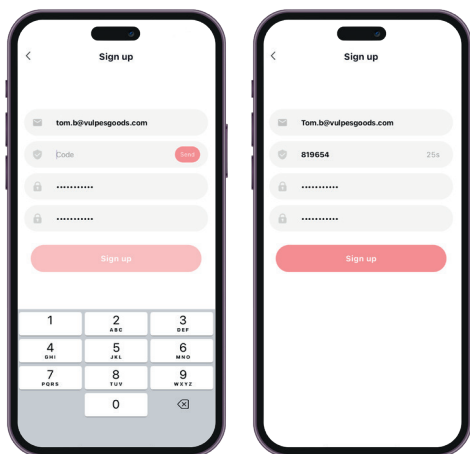Note: Both the email address and password are case-sensitive.

Step 3: Find and connect your breast pump

Make sure the breast pumps are turned on. In the app, click Add Device and ensure the pumps are within range.

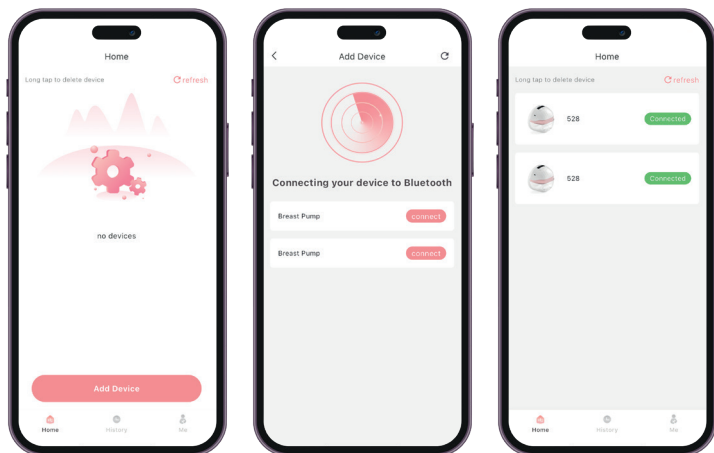
Note: Each pump must be connected separately to control them individually.

When a pump is connected, you'll see the message 528 Connected. Click on a connected pump to access the control panel.

In the Home screen, you can choose:

History: View your saved pumping history.

Me: Adjust your settings.


Step 4: Operate the pump and log milk amounts

After connecting the pumps, you will land on the control page. Click on a connected pump to open its menu. Here, you can:

Rename the pump, turn the pump on/off or pause it, select a mode, adjust the suction level, set the timer, and fill out the diary with the amount of milk pumped.

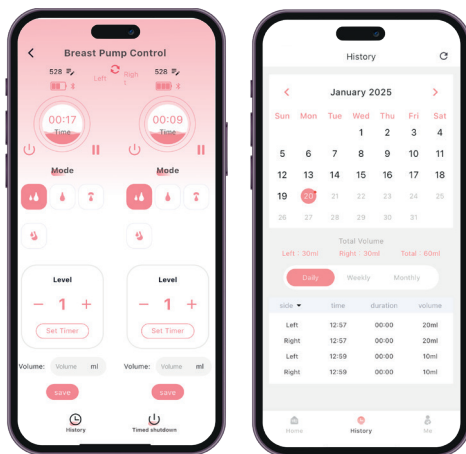Note: The amount of milk can only be logged when the pump is turned off.

WARNINGS

1. Use

- Use the breast pump exclusively for expressing breast milk. Other uses are considered improper use of the product.
- Consult a doctor or lactation consultant before using the breast pump, especially if you have medical questions or experience issues with breastfeeding.
- Ensure that the suction strength and speed are correctly adjusted to prevent discomfort and potential damage to the nipples or breasts.
- Do not use the breast pump on irritated or damaged skin. Wait until the skin has healed before using the breast pump again.
- Do not use the breast pump while sleeping or resting. Stay alert and aware during pumping.
- Pumping at regular intervals helps to maintain a consistent milk supply.
- Vary the modes and settings to prevent habituation. This can improve output.

2. Safety and Maintenance

- Inspect the breast pump and all parts for damage or wear before each use. Do not use damaged parts. Contact us immediately for defective parts.
- Only use the included charging cable. Using an incorrect power source can cause damage.
- Keep the motor and adapter dry and away from water or other liquids.
- Store the breast pump and accessories out of reach of children and pets.

- Stop pumping and consult a healthcare provider if you experience pain or discomfort while using the breast pump.
- Ensure the motor remains free from dust and milk residue. This can affect the durability of the motor.
- Ensure the motor is always free from milk, water, dust, dirt, and residue, especially on the underside of the motor, as this can negatively impact the motor's performance and durability. Damage from milk, water, dust, dirt, residue, improper handling, or negligent cleaning and usage is not covered under warranty.
- Do not drop the motor. Impact damage can cause serious internal damage and is not covered under warranty.
- Regularly check the silicone parts (like valves) and replace them if necessary, as they can wear out and affect suction power.
- Do not use chemicals or aggressive cleaning agents on the parts. This can be harmful to the product and the user.
- Always place the motor with the buttons facing up. The motor should be standing and not lying down. Placing the motor down can affect the durability of the pump.

3. Cleaning and Storage

- Thoroughly wash your hands before assembling or using the breast pump.
- Clean the breast pump and accessories after each use according to the instructions in the manual to prevent infections.
- Do not put the parts in the dishwasher; this can cause damage that is not covered under warranty.
- Store breast milk in clean bottles, bags, or well-sealed containers and at the correct temperature. Label these with the date of expression.
- Never heat breast milk in a microwave. Use a bottle warmer or warm water bath.
- Ensure all parts are completely dry before reassembling the breast pump to prevent electrical issues or mold growth.
- Do not use brushes or sharp objects to clean the valves, as this may cause damage. Do not pierce the valves and avoid applying excessive pressure.

4. Warranty and Contact

- In case of a defect, contact Vulpes Goods directly. Do not attempt to repair the device yourself and stop using it immediately.
- Water and impact damage are not covered under warranty. Keep the motor dry and do not drop it.
- Failure to follow the instructions, cleaning guidelines, and warnings may result in the warranty being voided. Engels

CLEANING & MAINTENANCE

Before cleaning and disinfecting, please remove all accessories and the motor. The pump motor contains electronics that should be kept dry. Use a clean cloth to wipe the pump motor. Any other cleaning method will be classified as misuse under the warranty.

Breast pumps are personal care items intended for a single user only and should never be rented or shared. Immediately separate and wash all parts that come into contact with the breast and breast milk after use to avoid residue buildup and prevent bacteria growth.

Cleaning

Non-washable Parts:

The motor must never come into contact with milk, water, dust, dirt, and residue, especially on the underside of the motor. The pump must remain dry and clean. Use a clean, dry cloth to wipe the main pump unit. Keep the motor dry and free of residue. Any cleaning method outside of these instructions may void the warranty due to improper use.

Washable parts:

This includes the gasstorage, the duckbill valve, the insert funnel, the sealing cap, the breastshield, and the collection cup. Cleaning the washable parts that come into contact with milk prevents the buildup of dried milk residue, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

Supplies needed:

Mild dish soap, clean discloth, clean wash basin & clean water

1. Rinse all separated parts that come into contact with the breast and breast milk in water at 25-30°C (77-86°F) to remove any milk residue.
2. Once rinsed, place the parts that have touched the breast or breast milk in a clean wash basin designated only for washing infant feeding equipment.
3. Let all washable parts soak in soapy water at 40-60°C (104-140°F) for 3-5 minutes. Wash each part with a clean dishcloth or sponge.
4. Place the cleaned parts on a clean surface or towel and allow them to air dry. Store dry parts in a clean, cool place when not in use. Do not store wet or damp parts.

Sanitizing

Non-washable parts:

The motor must never come into contact with milk, water, dust, dirt, and residue, especially on the underside of the motor. The pump must remain dry and clean. Use a clean, dry cloth to wipe the main pump unit. Keep the motor dry and free of residue. Any cleaning method outside of these instructions may void the warranty due to improper use.

Washable parts:

This includes the gasstorage, the duckbill valve, the insert funnel, the sealing cap, the breastshield, and the collection cup. Cleaning the washable parts that come into contact with milk prevents the buildup of dried milk residue, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

Supplies needed:

Clean pot for +100°C / +212°F water, clean water & tongs

1. Fill a pot with enough water to cover all parts. Bring the water to a boil. Place parts in water at +100°C / +212°F for 5-10 minutes, ensuring that the parts do not touch the sides of the pot or each other to avoid melting together.
2. Allow the water to cool and gently remove the parts from the water using tongs. Be cautious as both the water and the parts can be hot.
3. Place the sanitized parts on a clean surface or towel and allow them to air dry. Store dry parts in a clean, cool place when not in use. Do not store wet or damp parts.

**Replace washable parts after signs of wear and tear.
Replace washable parts every 1 to 3 months for hygiene.**

FAQ

Q. My breast pump has low suction?

A. The gas storage and pump unit are not properly installed in place. When the pump is installed in the gas storage unit, it must be fully depressed.

A. The duckbill valve is not installed in place or is damaged. Check the part for proper placement and possible damage.

Q. The milk flows back.

A. The gas storage is damaged or deformed, the milk flows back into the pump. In this case, stop using it immediately. Replace the gas storage to restore the connection.

A. The duckbill valve contains a crack. This allows the milk to flow back. Check the valve for cracks, damage and proper placement.

Q. Pumping hurts / pumping fails.

A. You may have a breast congestion or blocked milk duct. Ask a doctor to massage the breast.

A. Get the right breastplate size. Choosing the right size is important for a good fit.

A. The pump level used is too strong. Start the pumping session from the lower level and build up slowly.

Q. The milk remains on the breastplate.

A. Check that you are using the right breast shield reducer. Sometimes a size that is too small or too large can cause the milk not to flow properly.

Q. My breast pump is not charging properly?


A. Make sure you use the USB cable provided. Connect it to a suitable adapter with the correct voltage.

PRODUCT PARAMETERS

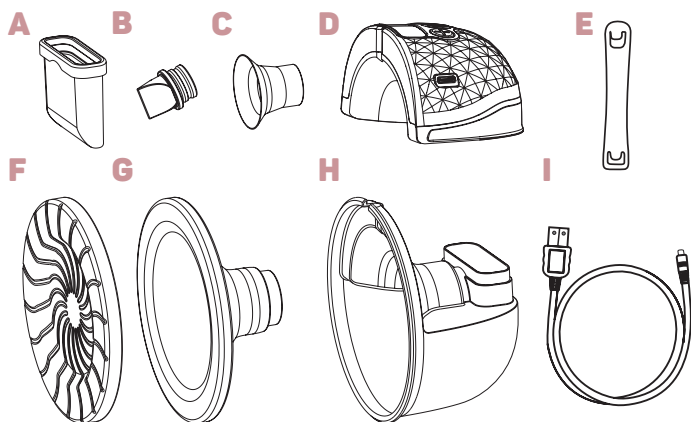Model:	VG-528
Weight (1 pump):	255Gr
Product:	11.5cm x 7cm x 11.7cm (LxWxH)
Nominal power:	5W
Input:	100-240V, 50/60Hz
Output power:	5V  1A
Suction strength:	< 0.0035 MPa
Battery capacity:	1100 mAh
Modes:	4
Collection tray capacity:	150ML

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