VULPES GOODS
BABYCARE

ELECTRIC BREAST PUMP

PRO V2

**APPLICATION
HANDLEIDING • MANUAL**

WWW.VULPESGOODS.COM

NEDERLANDS

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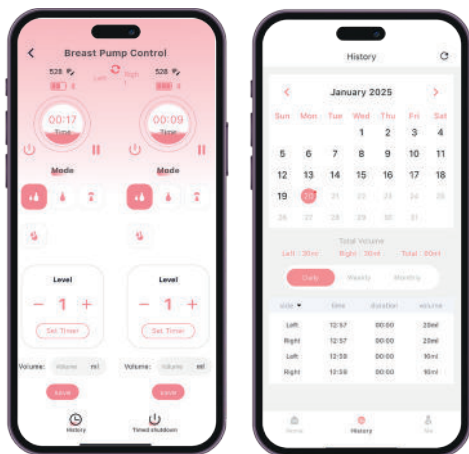
ENGLISH

Connecting to the app

To make pumping even more enjoyable, you can pair the PRO V2 with the EasyPump app via bluetooth. From the app, you can change the mode but it is also possible to keep track of the yield. Follow the steps below to use the app correctly.

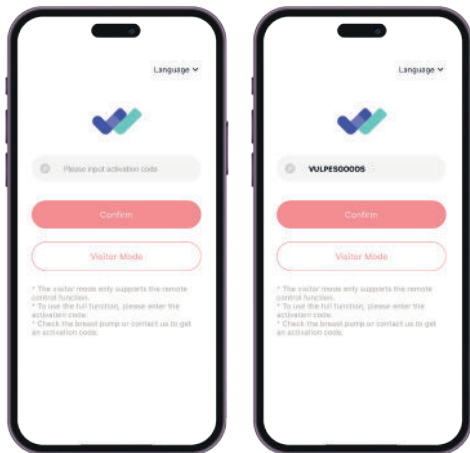
Using the app

- The app can only be used if the breast pump is on and connected.
- Turning the breast pump on from the app is not possible, switching it off is. Controlling the breast pump is possible via the app after switching it on.
- A minimum of 5 seconds must have been pumped, otherwise the session cannot be closed.
- Always select the right pump to operate. We recommend you rename the breast pumps.
- When pumping with two pumps and off is pressed. Will stop only the selected breast pump.
- If pumping is resumed from pause mode, there may be a second delay after pressing play.

Step 1: Open the app and connect

Open the app and enter VULPESGOODS as the activation code to log in.

Note: The code is case-sensitive.


Step 2: Log in or create an account

After entering the activation code, you can create an account:

Enter your email address and click Send to receive a verification code. Check your email and enter the received code in the Code field.

Set a password to complete your account setup. You can now log in using this email address and password.

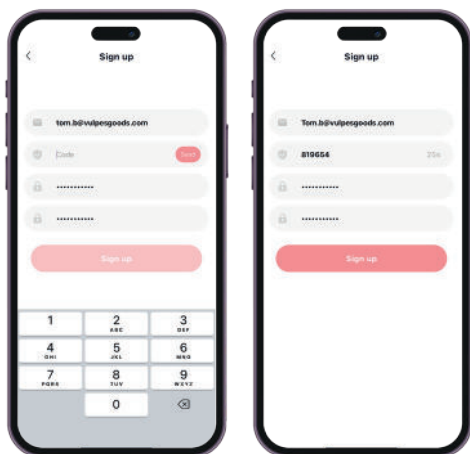Note: Both the email address and password are case-sensitive.

Step 3: Find and connect your breast pump

Make sure the breast pumps are turned on. In the app, click Add Device and ensure the pumps are within range.

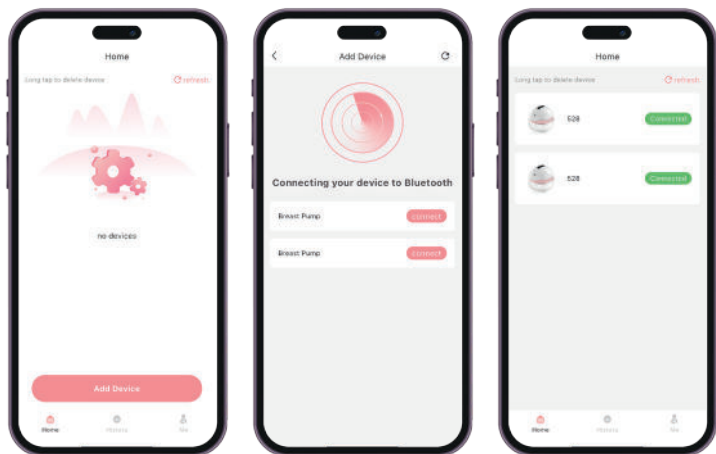
Note: Each pump must be connected separately to control them individually.

When a pump is connected, you'll see the message 528 Connected. Click on a connected pump to access the control panel.

In the Home screen, you can choose:

History: View your saved pumping history.

Me: Adjust your settings.

Step 4: Operate the pump and log milk amounts

After connecting the pumps, you will land on the control page. Click on a connected pump to open its menu. Here, you can:

Rename the pump, turn the pump on/off or pause it, select a mode, adjust the suction level, set the timer, and fill out the diary with the amount of milk pumped.

Note: The amount of milk can only be logged when the pump is turned off.